wwellion®
WAVE *professional*

HANDBUCH

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OWNERS GUIDE

TABLE OF CONTENTS

INTRODUCTION	17
ABOUT BLOOD PRESSURE	18
What is blood pressure?	18
What is hypertension and how is it controlled?	18
Why measure blood pressure at home?	18
WHO blood pressure classification	18
Blood pressure variations	19
NOTES ON SAFETY	19
Explanation of the icons	19
PRECAUTIONS BEFORE USE	21
PARTS IDENTIFICATION	21
INSERT OR REPLACE BATTERIES	22
Symbols on display	22
Accessory	22
ABNORMAL HEARTBEAT SIGNAL (♥)	23
TIME SETUP	24
SETUP UNIT mmHg/kPa	24
WHO BLOOD PRESSURE CLASSIFICATION DISPLAY	24
ATTACHING THE ARM CUFF	25
How to take proper measurements	25
HOW TO MEASURE BLOOD PRESSURE	25
Read memory	25
Delete Memory	26

CARE AND MAINTENANCE	26
TROUBLESHOOTING	27
SPECIFICATIONS	28
STATEMENT	28
FEATURE OF THE PRODUCT	28
REQUESTS FROM MANUFACTURER REGARDING EMC STATEMENT	29
MANUFACTURER'S WARRANTY	29

INTRODUCTION

The Wellion WAVE professional Blood Pressure Monitor uses the oscillometric method of blood pressure measurement.

The electronic Wellion WAVE professional Blood Pressure Monitor for automatic measurement is intended for use by medical professionals or at home to monitor and display diastolic and systolic blood pressure and pulse rate, with an arm cuff around the left upper arm according to the instruction in the section "ATTACHING THE ARM CUFF".

The product complies with the electromagnetic compatibility requirements of EN 60601-1-2 and safety standards of EN 60601-1 and performance of IEC 80601-2-30 as specified in EEC directive 93/42/EEC.

ABOUT BLOOD PRESSURE

1. What is blood pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands.

Blood pressure is measured in millimetres of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

2. What is hypertension and how is it controlled?

Hypertension is an abnormally high arterial blood pressure. If this is left unattended it can cause many health problems including stroke and heart attack.

Hypertension can be controlled by altering lifestyle, avoiding stress and with medication under a doctor's supervision.

To prevent hypertension or to keep it under control:


- Do not smoke
- Reduce salt and fat intake
- Maintain proper weight
- Exercise regularly
- Have regular physical checkups

3. Why measure blood pressure at home?

Blood pressure measured at a clinic or doctor's office may cause apprehension and produce an elevated reading, 25 to 30 mmHg higher than the value measured at home. Home measurement reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.

4. WHO blood pressure classification

Standards for assessment of high blood pressure, without regard to age, have been established by the World Health Organization (WHO), and shown in chart beside.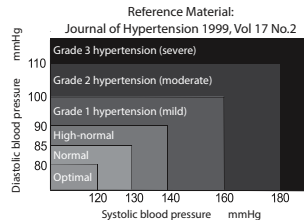

5. Blood pressure variations


An individual's blood pressure varies on daily and seasonal basis. It may vary by 30 to 50 mmHg due to various conditions during the day.

In hypertensive individuals, variations are even more pronounced.

Normally, the blood pressure rises while at work or play and falls to its lowest levels during sleep. So, do not be overly concerned by the results of one measurement.

Take measurements at the same time every day using the procedure described in this manual, and know your normal blood pressure. Many readings give a more comprehensive blood pressure history.

Be sure to note date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.

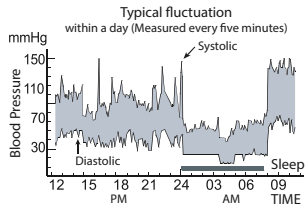NOTES ON SAFETY

The warning signs and sample icons shown here are listed for your safety and the correct use of the unit, so as to prevent injuries or damages to the device.

The icons and meanings are as follows:

Explanation of the icons

The icon indicates prohibitions (what you should not do). Matters involving actual prohibitions are indicated by text or pictures in or near. The left icon refers to “general prohibition”.

The icon indicates something that is compulsory (what must always be observed). Matters involving actual compulsory actions are indicated by text or pictures in or near. The left icon refers to “general compulsion”.

The icon indicates something can't be disassembled or “Don't disassemble”.

Type BF Applied part

Attention and read before use

Please refer to the instructions for use

Marking of electrical and electronic equipment in accordance with Article 11(2) of Directive 2002/96/EC (WEEE)

Indicates a medical device that needs to be protected from moisture.



<p>Patient must follow doctor's instruction and should not perform self-judgment and self-treatment by the measuring result. Self-diagnosis of measured results and treatment are dangerous. The device should not be used to judge illness, first aid measures and continuous monitoring of blood pressure. This device cannot be used for patient transport and surgical care. It can be used in household or fixed places only. Please press the "ON/OFF" button to stop the device from working when you feel uncomfortable with the arm, or if the air is inflating abnormally without stop.</p>	
<p>Do not let a child below the age of 12 years and people who can't express one's intention use the device. When it is used by people at the age of 12-18 years, an adult should accompany it, as it may cause an accident or trouble.</p>	
<p>Do not use the device for purposes other than measuring blood pressure, as it may cause an accident or trouble.</p>	
<p>Please do not use mobile phones around the device. Please do not use the device around a magnetic field.</p>	
<p>The device is prohibited from being used during movement.</p>	
<p>Do not use the equipment in outdoor or shower rooms.</p>	
<p>Stop using the equipment immediately, if it is in contact with water.</p>	
<p>Do not disassemble, repair, or remodel the main unit or the cuff of the blood pressure monitor. This will cause the unit to function erroneously.</p>	

PRECAUTIONS BEFORE USE


1. If you are taking medication, consult with your doctor to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your doctor.
2. For people with irregular or unstable peripheral circulation problems due to diabetes, liver disease, hardening of the arteries, etc., there may be fluctuation in blood pressure values measured at the upper arm versus at the wrist.
3. Measurements may be impaired if this device is used near televisions, microwave ovens, X-ray, mobile phone equipment or other devices with strong electrical fields.
To prevent such interference, use the monitor at a sufficient distance from such devices or turn them off.
4. Before using you should wash your hands.
5. Do not measure on the same arm which uses simultaneously another medical monitoring ME Equipment, otherwise it could cause loss of function.
6. Consult your doctor if unexpected readings are obtained, also please refer to "Trouble shooting" of the manual.
7. The reading at home is probably a little lower than measured in the hospital due to the steady mood at home.
8. Cuff pressure range 0-299mmHg

PARTS IDENTIFICATION

LCD Display
(Date and Time,
Systolic Blood Pressure,
Diastolic Blood Pressure,
Pulse/min)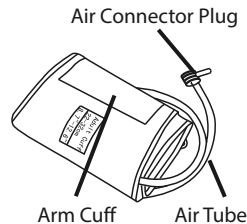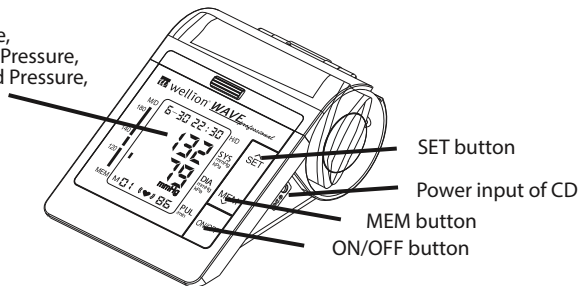


Symbols on display

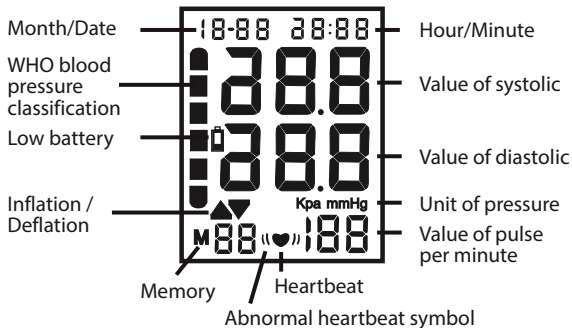Accessory

Manual

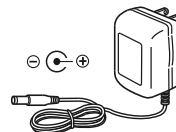
AC Adapter (optional)

Use exclusively a safety-proofed AC adapter which can be purchased from the manufacturer of the blood pressure meter.

INSERT OR REPLACE BATTERIES

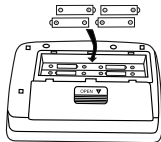1. Remove the battery cover.
2. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
3. Close the battery cover. Use only LR6, AA batteries.

Disposal of empty battery to the authorized collecting party subject to the regulation of each individual territory.

 Caution:

- Insert the batteries as shown in the battery compartment. If not, the device will not work.
- When „LOW BATTERY“ symbol  blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- „LOW BATTERY“ symbol  does not appear when the batteries run out.
- Please ensure to distinguish positive polar (+) and negative polar (-) of batteries when replacing batteries.
- Batteries, which have a fluid on the surface or have been modified, must not be inserted into the products.
- Battery short circuit must be prevented.
- Battery life varies with the ambient temperature and may be shorten at low temperatures.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.
- Used batteries may leak and damage the main unit. Please observe the following points:
 - If you are not going to use the unit for a long period of time (approximately three months or more), remove the batteries.
 - Replace empty batteries with their polarities in the correct direction.


ABNORMAL HEARTBEAT SIGNAL (♥)

When the detects an irregular rhythm two or more times during the measurement, the Abnormal heartbeat symbol will appear on the display with the measurement values.

An Abnormal heartbeat rhythm is defined as a rhythm that is 25% less or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure.

TIME SETUP

1. Press and hold the "SET" button until the year number displays and flashes on the LCD to enter setting mode.
2. Press the "MEM" button to adjust the year, then press the "SET" button again to save your setting and enter the month setting mode.
3. Following the same steps to adjust day/hour/minute until the setting is completed.

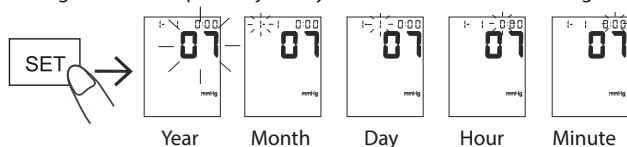SETUP UNIT mmHg/kPa

The results can be displayed as one of two different units: mm Hg (mmHg) or kPa (kPa). The device's preset is mmHg. To change the measuring unit, press and hold the "ON/OFF" button for five seconds, then press the "MEM" button. Please note that all memorized results change their unit as well.

WHO BLOOD PRESSURE CLASSIFICATION DISPLAY

Diastolic blood pressure Reference material:
journal of hypertension 1999. vol 17 No.2

- Grade 3 hypertension (severe)
- Grade 2 hypertension (moderate)
- Grade 1 hypertension (mild)
- High-normal
- Normal
- Optimal

ATTACHING THE ARM CUFF

1. Wrap the arm cuff around the upper arm, about 2-3 cm above the elbow, as shown. Place the cuff direct on the skin, as clothing may cause a faint pulse and result in a measurement error.
2. Constriction of the upper arm, caused by rolling up a shirtsleeve, may prevent accurate readings.
3. Secure the arm cuff with the hook-and-loop strip in such a way that it lies comfortably and is not too tight. Lay the arm on the table (palm upwards) so that the arm cuff is at the same height as the heart. Make sure that the tube is not kinked.


(2-3)cm

How to take proper measurements




For best accuracy in blood pressure measurement:

- Recommended patient position: Sit comfortably at a table. Rest your arm on the table.
- Relax for about 5 to 10 minutes before making a measurement.
- Remain still and keep quiet during measurement.
- Do not measure right after physical exercise or a bath.
- Measure your blood pressure at about the same time every day.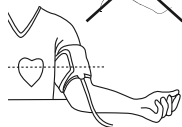

HOW TO MEASURE BLOOD PRESSURE

1. Set up the arm cuff to your upper arm as described in the previous section of "ATTACHING THE ARM CUFF"
2. Press the "ON/OFF" button, all icons automatically appear two seconds on the display, then the device switches to measurement and displays "00".
3. When the measurement starts, the cuff in the strap will automatically inflate. The inflating icon will flash on the LCD display. When the measurement is completed, the results will be displayed on the LCD display.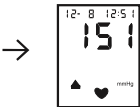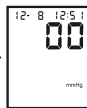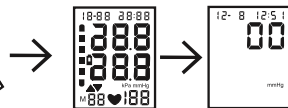


Read memory

Press the "MEM" button to scroll through the memorized results, starting with the latest measurement. Press "MEM" to go up, and "SET" to go down in the list of results.

Delete Memory

Enter the "READ MEMORY" mode, press and hold the "MEM" button for five seconds until the LCD displays "no". This indicates that all stored results have been deleted successful.

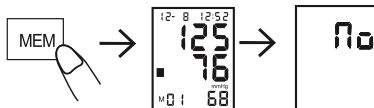CARE AND MAINTENANCE

To keep your digital blood pressure monitor in the best condition and to protect the unit from damage, follow the directions listed below:

- Keep the monitor in the storage case when not in use.
- Do not fold the arm cuff too tightly.
- Clean the monitor and cuff with a soft dry cloth.
- Do not use any abrasive or volatile cleaners.

Caution:

- Do not submerge the device or any of the components in water.
- Do not subject the monitor to extreme hot or cold temperatures, humidity or direct sunlight.
- Store the device and the components in a clean, safe location.
- Do not subject the monitor to strong shocks, such as dropping the unit on the floor.
- Remove the batteries if the unit will not be used for three months or longer.
- Always replace all the batteries with new ones at the same time.

Information:

- This product is designed for use over an extended period of time. It is generally recommended that it be inspected and calibrated every two years to ensure proper function and performance.

TROUBLESHOOTING

If you have trouble in using the product please check the following points first:

EN

Error display	Possible cause	How to correct
Nothing is displayed when you push the POWER button or battery icon  flashes	No battery installation	Insert batteries
	Battery worn out	Replace batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities
E0: device can't detect the signal of pulse	There may be many clothes between the cuff and upper arm. Please wear a thin shirt when measuring.	Constriction of the upper arm, caused by rolling up a shirtsleeve, may prevent accurate readings.
	The cuff may be worn in wrong direction, and the tube interface does not correspond to the arterial position.	
	If the E0 error code is not caused by the above mentioned reasons, the device is defect.	
E1: can't normally increase pressure	Check that your arm cuff has no air leakage	Replace arm cuff with new one
E3: inflate pressure too high		Re-measurement or have the device re-calibrated
E2/E4: shaking while measurement	Hand or body shaking while measurement	Keep static and correct gesture to measure again
Battery icon  displayed	Batteries have low power	Replace batteries and repeat the measurement
The systolic or diastolic pressure value is too high	The arm cuff was held lower than your heart	Keep correct position and gesture to measure again
	The arm cuff was not attached properly	
	You moved your body or spoke during measurement	
The systolic or diastolic pressure value is too low	The arm cuff was held higher than your heart	Keep correct position and gesture to measure again
	You moved your body or spoke during measurement	

SPECIFICATIONS

Measuring Method:	Oscillometric Measurement	Indication:	Digital LCD display
Measuring Range:	Pressure: 30 - 280 mmHg Pulse: 40 - 199 beats/minute	Accuracy:	Static Pressure: ± 3 mmHg Pulse: $\pm 5\%$
Memory:	90 Memories	Power supply:	4x 1,5V Batteries LR6 or AA
Operating condition:	+ 5°C - + 40°C; 15% RH - 93% RH; Atmospheric pressure: 70kPa - 106kPa	Storage condition:	- 20°C - + 55°C; 0% RH - 93% RH; Atmospheric pressure: 50kPa - 106kPa
Dimensions:	Approx: 165 (B) x 235 (H) x 100 (T) mm	Weight:	Approx: 750g, excl. batteries
Classification:	Type BF	Upper arm circumference:	22 - 42 cm
AC adapter:	Input: 100 – 240V 50/60Hz, 0,2A Output: 6V, 500 mA	Use exclusively a safety-proofed AC adapter which can be purchased from the manufacturer of the blood pressure meter.	

1. Type of protection against electric shock: INTERNALLY POWERED EQUIPMENT.
2. Degree of protection against electric shock: TYPE BF APPLIED PART.
3. Mode of operation: CONTINUOUS OPERATION.
4. Equipment not suitable for category AP&APG equipment use in presence.

STATEMENT

The system might not meet its performance specifications if stored or used outside the temperature and humidity as mentioned below:

- Operating conditions: +5°C~+40°C 15%RH~93%RH; 70kPa~106kPa
- Storage conditions: -20°C ~+55°C 0%RH~93%RH

FEATURE OF THE PRODUCT

1. Memory can store 90 measurements.
2. Large and clear LCD display.
3. WHO blood pressure classification display.
4. Easy to use: Press a button to automatically measure; Record the measurement values and measurement time.
5. Automatically turns off (within 1 minute) to save power.

REQUESTS FROM MANUFACTURER

- Make sure that the connection tube is not kinked before starting the measurement to avoid any injury to patient.
- For any patient, do not measure more than 3 times continuously, it should be at least above 5 minutes of interval rest between any two measurements, otherwise will cause exsanguination in the blood vessel.
- Do not measure your blood pressure over 6 times each day.
- Do not apply the cuff over a wound as this can cause further injury.
- Do not measure on the arm which is on the side of a mastectomy, otherwise it could cause injury.
- Observe the air pressure value from the LCD display.
- When measuring, the device cannot exceed 299 mmHg, otherwise please press the "ON/OFF" button to stop.
- Do not use force to bend the arm cuff or the air tube.
- Do not knock or drop the main unit.
- Always use the specified accessories in the manual, the use of other parts not approved by the manufacturer may cause faults or injuries.
- For service information, parts list etc., please contact the dealer.

REGARDING EMC STATEMENT

1. A statement to the effect that this equipment needs to be installed and put into service in accordance with the information provided in the ACCOMPANYING DOCUMENTS; and
2. a statement to the effect that wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept distanced away from the equipment. The distance is calculated by the MANUFACTURER from the 800 MHz to 2,5 GHz column of Table 5 or Table 6 of EN 60601-1-2:2015, as appropriate.

MANUFACTURER'S WARRANTY

The producer warrants Wellion WAVE professional blood pressure meter against defects in materials and workmanship for a period of two years from the date of purchase. The warranty is lost if the instrument is misused, poorly maintained or is tampered with. Liability under this warranty is limited to the repair of defective parts or - at the discretion of the producer - to the replacement of the instrument. The right to rescind the purchase agreement exists only if the replacement is also found to be defective. Claims other than these will not be dealt with. The warranty is not valid if the damage results from misuse, maltreatment, tampering, human error and the use of extreme force. This warranty is valid only if the date, the stamp and the signature of the dealer are recorded on the warranty card on the date of purchase. The warranty period is not to be extended by any claim made under this warranty.

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© MED TRUST, Wellion and WAVE are registered trademarks.